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The sea water used in **Sinomarin®** is collected from Cancale Bay at the coast of Brittany in France. The area is well-known for its open seas and strong tidal currents and thus constitutes sea water of outstanding quality rich in mineral salts and trace elements. The sea water quality is continuously monitored by the French Research Institute for Exploitation of the Sea (IFREMER) under the auspices of the French Government.

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Properties:

- ▶ Relieves nasal congestion by reducing edema of the nasal mucosa (via osmosis).
- ▶ Fluidifies mucus secretions making them easier to expel.
- ▶ Helps restore the natural function of the nasal mucosa.
- ▶ Cleanses the nasal cavities from accumulated mucus, crusts, dust, debris, pollutants, allergens and infectious agents.
- ▶ Moisturizes the nasal mucosa and relieves nasal dryness.

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Sinomarin® is recommended:

- ▶ For all cases of nasal and sinus congestion, e.g. rhinitis (infectious or allergic), sinusitis; for the common cold and flu.
- ▶ In cases where medicated decongestants are contraindicated (e.g. pregnancy, breastfeeding, thyroid disease, high blood pressure, heart conditions, diabetes).
- ▶ To enhance the efficacy of medicated decongestants and to reduce their use.
- ▶ For the prophylaxis of nasal & secondary ENT infections and complications e.g. otitis.
- ▶ To promote recovery following endonasal surgery.
- ▶ For daily hygiene (e.g. in polluted environments).
- ▶ For hydration in dry environments.

Composición: Sea water 70%, Purified water 30%, equivalente a 23 g/L NaCl (2.3%). Preservative-free. Drug-free. Chemical additives-free. 100% natural.

Descripción: 30 ml / 50 ml nasal pump sprays with preservative free pump, suitable for everyone including babies.

Recommended dosage: Use as per below, or as per physician's instructions.

+0 months: 1-2 sprays in each nostril, 2-3 times daily;

+6 months: 1-3 sprays in each nostril, 2-3 times daily.

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Instructions for use / Modo de empleo / Modo de utilizar

Adults & Children

- 1 Blow nose to clear out mucous before using **Sinomarin®**.
- 2 Remove protective cap.
- 3 Hold the spray bottle upright and spray a few times in order to activate the spray pump.
- 4 Bend neck gently forward over a wash basin and tilt head to the right.
- 5 Gently insert applicator into the left nostril in parallel with the nasal septum and press firmly.
- 6 Turn head to opposite side and repeat application in other nostril.
- 7 Remove spray from nostril, return to upright position while allowing solution to work for a few seconds and then blow the nose.
- 8 Repeat procedure if necessary.
- 9 Wash applicator well with warm water and wipe dry after each use.

Babies & children who require assistance

- 1 Wipe child's nose clean before using **Sinomarin®**.
- 2 Remove protective cap.
- 3 Hold the spray bottle upright and spray a few times in order to activate the spray pump.
- 4 Place baby in supine position turning its head to the right.
- 5 Gently insert applicator into the left nostril in parallel with the nasal septum and press firmly.
- 6 Turn head to opposite side and repeat application in other nostril.
- 7 Remove spray from baby's nostril, return baby to upright position while allowing solution to work for a few seconds and then wipe the nose clean.
- 8 Repeat procedure if necessary.
- 9 Wash applicator well with warm water and wipe dry after each use.

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Sinomarin® is a 100% natural, clinically proven nasal decongestant. It is a hypertonic sea water solution (2.3% NaCl) rich with the sea's beneficial ingredients (mineral salts and trace elements). **Sinomarin®** relieves nasal congestion and sinus pressure via the natural mechanism of osmosis. Moreover, **Sinomarin®** cleanses the nasal cavities, eliminating accumulated mucus, crusts, dust, debris, pollutants, allergens and infectious agents. Additionally, it helps moisturise the nasal mucosa.

The sea water used in **Sinomarin®** is collected from Cancale Bay at the coast of Brittany in France. The area is well-known for its open seas and strong tidal currents and thus constitutes sea water of outstanding quality, rich in mineral salts and trace elements. The sea water quality is continuously monitored by the French Research Institute for Exploitation of the Sea (IFREMER) under the auspices of the French Government.

Sinomarin® is free of drugs, preservatives and chemical additives.

Sinomarin® can be used alone, as a natural alternative to medicated decongestants (local corticosteroids, antihistamines, vasoconstrictors), or in combination with them to improve their efficacy. As such, it helps decrease medicated treatment duration and may lead to reduced overall drug intake.

Properties:

- Relieves nasal congestion by reducing edema of the nasal mucosa (via osmosis).
- Fluidifies mucus secretions making them easier to expel.
- Helps restore the natural function of the nasal mucosa.
- Cleanses the nasal cavities from accumulated mucus, crusts, dust, debris, pollutants, allergens and infectious agents.
- Moisturizes the nasal mucosa and relieves nasal dryness.

Sinomarin® improves quality of life by alleviating symptoms linked to nasal congestion (e.g. itchy nose, sneezing, cough, sinus and ear pressure, sleep disturbance).

Sinomarin® is recommended:

- For all cases of nasal and sinus congestion, e.g. rhinitis (infectious or allergic), sinusitis; for the common cold and flu.
- In cases where medicated decongestants are contraindicated (e.g. pregnancy, breastfeeding, thyroid disease, high blood pressure, heart conditions, diabetes).
- To enhance the efficacy of medicated decongestants and to reduce their use.
- For the prophylaxis of nasal & secondary ENT infections and complications, e.g. otitis.
- To promote recovery following endonasal surgery.
- For daily hygiene (e.g. in polluted environments).
- For hydration in dry environments.

Description:

Sinomarin® Children: 100 ml nasal spray bottle. Gentle mist spray with safety tip suitable for children and babies over 1 month.

*The product is also available in 50 ml & 125 ml presentations.

Sinomarin® Adults: 125 ml nasal spray bottle. Normal stream spray for adults and children over 6 years.

Sinomarin® E.N.T.: 200 ml nasal spray bottle. Dynamic Jet spray for adults and children over 12 years.

Composition: Sea water 70%, Purified water 30%, equivalent to 23 g/L NaCl (2.3%). Preservative-free. Drug-free. Chemical additives-free. 100% natural.

Recommended dosage:

Use as per below, or as per physician's instructions:

Sinomarin® Children: 1+ months: 1-3 sprays in each nostril daily; 6+ months: 1-2 sprays in each nostril, 2-3 times daily.

Sinomarin® Adults & Sinomarin® ENT: 1 to 3 sprays in each nostril, 2 to 3 times daily.

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Precautions: For nasal use only. Do not use same nasal applicator on more than one individuals. Do not use past expiry date. Keep out of reach of children. Do not use in case of hypersensitivity to sea water.

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Storage conditions: Store below 25 °C. Pressurized container. Protect from sunlight and sources of heat. Do not expose to temperatures above 50 °C. Do not puncture or incinerate container even when empty.

Other information: Inert propellant gas (nitrogen), not in contact with solution. Solution held in a separate container.

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Suitable during pregnancy and breastfeeding.

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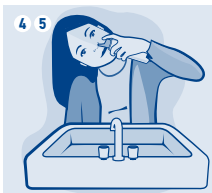
Instructions for use / Modo de empleo / Modo de utilizar

Adults & Children

- 1 Blow nose to clear out mucous before using **Sinomarin®**.
- 2 Break tamper-proof seal on bottle and remove protective cap.
- 3 Place nasal applicator on top of the bottle.
- 4 Bend neck gently forward over a wash basin and tilt head to the right.
- 5 Gently insert applicator into the left nostril in parallel with the nasal septum and press firmly.
- 6 Turn head to opposite side and repeat application in the other nostril.
- 7 Remove nasal applicator from nostril, return to upright position while allowing solution to work for a few seconds and then blow the nose.
- 8 Repeat procedure if necessary.
- 9 Remove nasal applicator from bottle, wash well with warm water and wipe dry after each use.

Babies & Children who require assistance

- 1 Wipe baby's nose clean before using **Sinomarin®**.
- 2 Break tamper-proof seal on bottle and remove protective cap.
- 3 Place nasal applicator on top of the bottle.
- 4 Place baby in supine position turning its head to the right.
- 5 Gently insert applicator into the left nostril in parallel with the nasal septum and press firmly.
- 6 Turn head to opposite side and repeat application in the other nostril.
- 7 Remove nasal applicator from baby's nostril, return baby to upright position while allowing solution to work for a few seconds and then wipe the nose clean.
- 8 Repeat procedure if necessary.
- 9 Remove nasal applicator from bottle, wash well with warm water and wipe dry after each use.

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Sinomarin® Babies is a 100% natural, clinically proven nasal decongestant. It is a hypertonic sea water solution (2.3% NaCl) rich with the sea's beneficial ingredients (mineral salts and trace elements).

Sinomarin® Babies relieves nasal congestion and sinus pressure via the natural mechanism of osmosis. Moreover, **Sinomarin® Babies** cleanses the nasal cavities, eliminating accumulated mucous, crusts, dust, debris, pollutants, allergens and infectious agents. Additionally, it helps moisturise the nasal mucosa.

Sinomarin® Babies can also be used in a nebulized form to help enhance respiratory function. In infants with acute bronchiolitis, nebulized **Sinomarin® Babies** may help reduce airway edema and thin airway mucus, thus decreasing airway obstruction.

The sea water used in **Sinomarin® Babies** is collected from Cancale Bay at the coast of Brittany in France. The area is well-known for its open seas and strong tidal currents and thus constitutes sea water of outstanding quality, rich in mineral salts and trace elements. The sea water quality is continuously monitored by the French Research Institute for Exploitation of the Sea (IFREMER) under the auspices of the French Government.

Sinomarin® Babies is free of drugs, preservatives and chemical additives.

Sinomarin® Babies can be used alone, as a natural alternative to medicated decongestants (local corticosteroids, antihistamines, vasoconstrictors), or in combination with them to improve their efficacy. As such, it helps decrease medicated treatment duration and may lead to reduced overall drug intake.

Properties:

- Relieves nasal congestion by reducing edema of the nasal mucosa (via osmosis).
- Fluidifies mucus secretions making them easier to expel.
- Helps restore the natural function of the nasal mucosa.
- Cleanses the nasal cavities from accumulated mucus, crusts, dust, debris, pollutants, allergens and infectious agents.
- Moisturizes the nasal mucosa and relieves nasal dryness.
- Helps alleviate symptoms linked to nasal congestion (e.g. itchy nose, sneezing, cough, sinus and ear pressure, sleep disturbance).
- In nebulized form, **Sinomarin® Babies** helps reduce airway edema and thins airway mucus, thus decreasing airway obstruction. It helps enhance respiratory function and prevent lower respiratory tract infections.

Sinomarin® is recommended:

- For the active removal of accumulated mucous and cleansing of the nasal cavities.
- For all cases of nasal and sinus congestion, e.g. rhinitis (infectious or allergic), sinusitis; for the common cold and flu.
- For the prevention of nasal & secondary ENT infections and complications e.g. otitis.
- For daily hygiene and prophylaxis.
- To promote recovery following endonasal surgery.
- For hydration in dry environments.
- In cases of reduced respiratory function or in acute bronchiolitis (in nebulized form).

Description: 18x5 ml single dose vials for infants and babies.

*Also available in packs containing different number of vials.

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Recommended dosage:

A few drops in each nostril, 2 to 3 times daily, or as per physician's instructions. In cases of thick and superfluous mucus, a larger volume of product may be safely introduced in each nostril.

In nebulized form, up to 5 ml 3 times daily or as per physician's instructions. **Sinomarin® Babies** can be combined with any type of nebulizing device available in the market.

Precautions: For nasal and respiratory (with nebuliser) use only. Each vial is intended for single use only. Do not use past expiry date. Keep out of reach of children. Do not use in case of hypersensitivity to sea water

Storage conditions: Store below 25 °C. Protect from sunlight.

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Instructions for use / Modo de empleo /Modo de utilizar

- 1 Wipe baby's nose clean before using **Sinomarin® Babies**.
- 2 Break-off one vial from strip.
- 3 Twist tip to open vial.
- 4 Place baby in supine position turning its head to the right.
- 5 Introduce tip of vial carefully into the left nostril and press to release the desired quantity.
- 6 Turn baby's head to opposite side and repeat application in the other nostril.
- 7 Repeat procedure if necessary.
- 8 Remove vial from baby's nostril, return baby to upright position while allowing solution to work for a few seconds and then wipe the nose clean.

In case of use with a nebulizing device, please follow the usage instructions of the corresponding device as provided by the manufacturer.

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EN Sinomarin® Plus with Algomer™ complex is a 100% natural spray that provides relief from nasal congestion, nasal dryness & irritation and other symptoms of allergic rhinitis, chronic or seasonal. Its unique –patent pending– formulation combines the natural decongestant action (osmosis effect*) of hypertonic sea water (2.3% NaCl) with the well-documented properties –beneficial to the nasal mucosa– of seaweed (algae) extracts** present in Algomer™ complex, in addition to Dexpantenol. Sinomarin® Plus cleanses the nasal cavities helping to eliminate allergens (mechanical action) that may worsen allergic rhinitis symptoms.

The seawater used in Sinomarin® Plus is collected from Cancale Bay at the coast of Brittany in France. The area is well-known for its open seas and strong tidal currents, and provides sea water of outstanding quality, rich in the sea's mineral salts and trace elements. The sea water quality is continuously monitored by the French Research Institute for Exploitation of the Sea (IFREMER) under the auspices of the French Government.

Algomer™ complex is a unique combination of 100% natural ingredients extracted from *Undaria pinnatifida* and *Spirulina platensis* algae (seaweed). Both algae have long been used in Oriental medicine to promote health and treat multiple conditions, and are part of the Far-Eastern diet. Formulated specifically for nasal use, Algomer™ complex is composed of a blend of natural algae components, namely, phycocyanin and a series of polysaccharide complexes including fucoidans. These components form a protective layer that encapsulates, traps and prevents contact and/or entry of harmful external agents (e.g. allergens, pollutants, viruses, bacteria, etc.) within the nasal mucosa, helping to reduce symptoms of allergic rhinitis and strengthen its natural function. *Undaria pinnatifida* and *Spirulina platensis* algae are harvested from the most pristine waters of Patagonia in Argentina and Brittany in France, respectively, and are certified organic.

Dexpantenol –also known as provitamin B5– is a 100% natural ingredient well-known for its soothing and moisturizing properties. It relieves itching and local irritation and moisturises the nasal mucosa.

Sinomarin® Plus can be used alone, as a natural alternative to anti-allergic treatments (local corticosteroids, antihistamines) or in combination with them to help enhance their efficacy.

Properties:

- ▶ Relieves allergic rhinitis symptoms including nasal congestion, sinus pressure, runny nose and sneezing.
- ▶ Relieves local irritation and soothes the nasal mucosa.
- ▶ Cleanses the nasal cavities eliminating allergens.
- ▶ Moisturises the nasal mucosa, relieving nasal dryness.
- ▶ Helps promote natural recovery of the nasal mucosa.

Sinomarin® Plus is recommended:

- ▶ For the natural relief of nasal symptoms associated with seasonal (e.g. hay fever) or chronic allergies such as blocked nose, sinus pressure, runny nose, sneezing, itchy nose, local irritation.
- ▶ To eliminate mucus and mucus-trapped allergens (mechanical action) that may worsen allergic rhinitis symptoms.
- ▶ In cases where prescribed anti-allergic agents are contraindicated.
- ▶ To help enhance the efficacy of prescribed anti-allergic drugs.

Composition: Sea water (equivalent to 2.3% NaCl), Purified water, Algomer™ complex (*Undaria pinnatifida* & *Spirulina platensis* extracts), Dexpantenol.

Sinomarin® Plus is free of antihistamines & steroids, preservatives and chemical additives. It is 100% natural and does not cause side effects such as drowsiness.

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Recommended dosage:
For adults & children over 6 years.
6-12 years: 1 to 2 sprays in each nostril up to 3 times daily.

>12 years: 2 to 4 sprays in each nostril up to 5 times daily.

Precautions: For nasal use only. Do not use same nasal applicator on more than one individuals. Do not use past expiry date. Keep out of reach of children. Do not use if sensitive to any of the components.

Advice: A mild and temporary irritation may be experienced at the beginning of use.

Storage conditions:

Store below 25 °C. Protect from sunlight.

Other information:

Special valves prevent contamination of the solution once the spray has been opened and thereby the necessity to use preservatives.

Sinomarin® is a complete line of hypertonic sea water solutions covering the needs of all ages from newborns to adults.

Sinomarin® products are CE-marked Medical Devices conforming to the requirements of Directive 93/42 of the European Union.

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Instructions for use / Instrucciones de uso

- 1 Blow nose to clear out mucus before using **Sinomarin® Plus**.
- 2 Remove protective cap.
- 3 Hold the spray bottle upright and spray a couple of times in order to activate the spray pump.
- 4 Bend neck gently forward over a wash basin and tilt head to the right.
- 5 Gently insert applicator into the left nostril, in parallel with the nasal septum, and press firmly.
- 6 Turn head to opposite side and repeat application in other nostril.
- 7 Remove spray from nostril, return to upright position while allowing solution to work for a few seconds and then blow the nose.
- 8 Repeat procedure if necessary.
- 9 Wash applicator well with warm water and wipe dry after each use.

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* Natural process characterized by water movement out of the swollen nasal tissues due to the local difference in salt (NaCl) content induced by nasal application of **Sinomarin®**.

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Sinomarin® Plus is covered by an international patent application (pending) filed by Gerolymatos International S.A., E.U.

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