**CLINICALLY TESTED  
PATENTED PRODUCT**

**100% NATURAL NASAL DECONGESTANT  
HYPERTONIC SEA WATER SOLUTION**

**Sinomarin® is a 100% natural, clinically proven<sup>1,2</sup> nasal decongestant.** It is a hypertonic sea water solution (2.3% NaCl) rich with the sea's beneficial ingredients (mineral salts and trace elements). **Sinomarin®** relieves nasal congestion and sinus pressure via the natural mechanism of osmosis. Moreover, **Sinomarin®** helps moisturize and cleanse the nasal cavities, eliminating accumulated mucous, crusts, dust, debris, pollutants, allergens and infectious agents.

The sea water used in **Sinomarin®** is collected from Cancale Bay at the coast of Brittany in France. The area is well-known for its open seas and strong tidal currents and thus constitutes sea water of guaranteed quality. The water is closely monitored by the French Water Authorities and satisfies the standards imposed by the French Ministry of Health.

**Sinomarin®** is free of drugs, preservatives and chemical additives. No side effects have been reported with the use of **Sinomarin®** such as drying of the nasal cavities, habit formation or rebound effect. **Sinomarin®** can be used as often and for as long as is required without limitation.

**Sinomarin®** can be used alone, as a natural alternative to medicated decongestants (local corticosteroids, antihistamines, vasoconstrictors), or in combination with them to improve their efficacy. As such, it helps decrease medicated treatment duration and may lead to reduced overall drug intake.

**Sinomarin®** is a complete line of hypertonic-only sea water solutions covering the needs of all ages from newborns to adults:

- ▶ **Sinomarin® Bebés 5 ml viales**  
For infants and babies.
- ▶ **Sinomarin® Children 100 ml**  
Gentle mist spray with safety tip - for babies over 6 months and children.
- ▶ **Sinomarin® Adults 125 ml**  
Normal stream spray - for adults and children over 6 years.
- ▶ **Sinomarin® ENT 200 ml**  
Dynamic jet spray - for adults and children over 10 years.
- ▶ **Sinomarin® Mini Spray 30 ml & 50 ml**  
Pump sprays - for adults, children and babies over 6 months.

**Properties:**

- ▶ Relieves nasal congestion by reducing edema of the nasal mucosa (via osmosis).
- ▶ Fluidifies mucous secretions.
- ▶ Helps restore the natural function of the nasal mucosa.
- ▶ Cleanses the nasal cavities from accumulated mucous, crusts, dust, debris, pollutants, allergens and infectious agents.
- ▶ Moisturizes the nasal mucosa and relieves nasal dryness.

**Sinomarin®** improves quality of life by alleviating symptoms linked to nasal congestion (e.g. itchy nose, sneezing, cough, sinus and ear pressure, sleep disturbance).

**Sinomarin® is recommended:**

- ▶ For all cases of nasal and sinus congestion, e.g. rhinitis (infectious or allergic), sinusitis; for the common cold and flu.
- ▶ In cases where medicated decongestants are contraindicated (e.g. pregnancy, breastfeeding, thyroid disease, high blood pressure, heart conditions, diabetes).
- ▶ To enhance the efficacy of medicated decongestants and to reduce their use.
- ▶ For the prophylaxis of secondary ENT infections and complications e.g. otitis.
- ▶ To promote recovery following endonasal surgery.
- ▶ For daily hygiene (e.g. in polluted environments).
- ▶ For hydration in dry environments.

**Composition:** Sea water 70%, Purified water 30%, equivalent to 23 g/L NaCl (2.3%). Preservative-free. Drug-free. Chemical additives-free. 100% natural.

**Description:** 30 ml & 50 ml nasal pump spray bottles. Metered-dose sprays with Advanced Preservative Free™ pump, for adults, children and babies over 6 months, with anatomically designed nasal applicator.

**Recommended dosage:** 1 to 3 sprays in each nostril, 2 to 3 times daily, or as per physician's instructions.

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CLINICALLY TESTED  
PATENTED PRODUCT

100% NATURAL NASAL DECONGESTANT  
HYPERTONIC SEA WATER SOLUTION

### Instructions for use / Modo de empleo / Modo de utilizar

#### Adults & Children

- Hold the spray bottle upright and spray a few times in order to activate the spray pump.
- Blow nose to clear out mucous before using **Sinomarin**.
- Remove protective cap.
- Bend neck gently forward over a wash basin and tilt head to the right.
- Gently insert applicator into the left nostril parallel to the nose and press firmly.
- Turn head to opposite side and repeat application in other nostril.
- Remove spray from nostril, return to upright position while allowing solution to work for a few seconds and then blow the nose.
- Repeat procedure if necessary.
- Wash applicator well with warm water and wipe dry after each use.

#### Babies & children who require assistance

- Hold the spray bottle upright and spray a few times in order to activate the spray pump.
- Wipe child's nose clean before using **Sinomarin**.
- Remove protective cap.
- Place baby in supine position turning its head to the right.
- Gently insert applicator into the left nostril parallel to the nose and press firmly.
- Turn head to opposite side and repeat application in other nostril.
- Remove spray from baby's nostril, return baby to upright position while allowing solution to work for a few seconds and then wipe the nose clean or use an aspirating device to remove mucous.
- Repeat procedure if necessary.
- Wash applicator well with warm water and wipe dry after each use.

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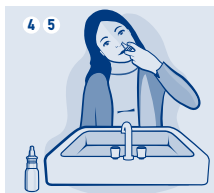
**Precautions:** For nasal use only. Do not use on more than one individual. Do not use past expiry date. Keep out of reach of children.

**Advice:** A mild and temporary stinging sensation or temporary irritation may be experienced at the beginning of use.

**Storage conditions:** Store below 25 °C. Protect from sunlight.

**Suitable during pregnancy and breastfeeding.**

**Sinomarin**® products are CE-marked Medical Devices conforming to the requirements of Directive 93/42 of the European Union.

Also available in the **Sinomarin**® product range:

► **Sinomarin**® EAR CARE Children 30 ml

Ultra-gentle pump spray with gentle safety tip for children over 3 years.

► **Sinomarin**® EAR CARE Adults 30 ml

Pump spray with gentle safety tip for adults and children over 12 years.

**Sinomarin**® EAR CARE is ideal for cleansing the ear canal by gently washing away excess ear wax. Through regular use it prevents the accumulation of ear wax and ensures good ear hygiene.

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1. Freche C. et al., "Usefulness of hypertonic sea water (Sinomarin®) in rhinology", Revue Officielle de la Société Française de ORL, 50(4), 1998. 2. González G. et al., "A clinical study conducted in 2007 with Mexican patients: an investigational, prospective, longitudinal, comparative, multicentre, open-label study on the efficacy and tolerability of Sinomarin Spray for the treatment of rhinitis", Journal of the Federation of Otolaryngological and Societies of the Mexican Republic (FESORMEX), 2008.